



TASSO-SST

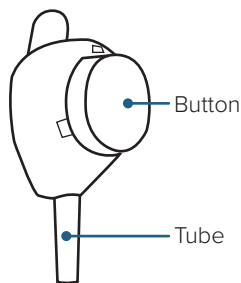

VIDEO


FIRST TIMER USERS

Point your phone's camera here to watch:

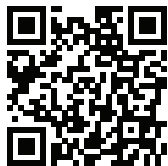

or go to
www.tassoinc.com/tasso-sst-video

PREPARE

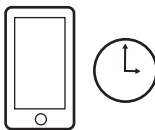

1. Get a timer.

You'll need it in step 9.




2. Rub arm quickly and firmly just below the shoulder.

Rub until it's very warm to help your blood flow.




3. Clean arm with alcohol pad.

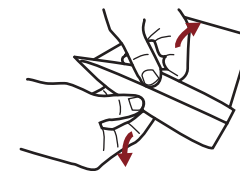


4. Open device pouch by pulling apart white and clear layers.

COLLECT

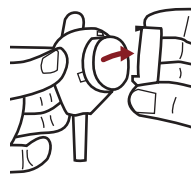

5. Remove clear plastic clover over the red button.

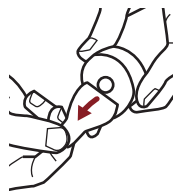

6. Peel paper tab behind the red button.

Keep tube pointing down.




7. Stick device to shoulder.

Do not remove once it's on.



8. Press button quickly and firmly until it can't go any farther. Wait two seconds then let go.

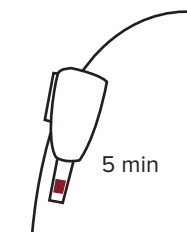

9. Start a five minute timer.

Keep arm at your side. You won't see blood right away.



10. After five minutes, peel off device.


No need to fill the tube completely.

RETURN



11. Remove tube by firmly twisting a quarter turn and pulling down.

This may take a bit of finger strength.

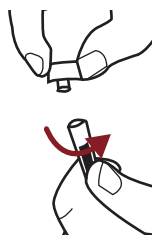


12. Twist cap fully onto the tube.



13. Place tube in the clear return bag and seal.


Leave the absorbent pad in the bag.

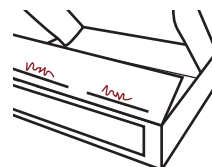

14. Write collection date and time on the inside flap of the box.



15. Place bag in box and seal with the strip.

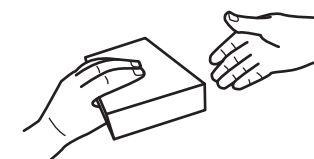

16. Return box within 24 hours of packing.

Throw out used Button in standard household trash.

**INSTRUCTIONS FOR USE
AGES 4 AND OLDER**



**TASSO ONDEMAND
TASSO-SST**

For investigational use only.



Contact us for help:
1-408-708-8451

Tasso, Inc.
1631 15th Ave. W., Suite 105
Seattle, WA 98119



PN0101-01, A2

WARNINGS

Prescription use only.
Use before the expiration date affixed to the box.
One-time use only.

Keep out of reach of children.

Minor bruising may occur around the sample collection site.

For external use only.

Intended for self-use or use by a healthcare provider.

Use while seated as fainting may occur with any blood sampling procedure.

INTENDED USE

TASSO-SST is a single use blood collection device that is intended for the self-collection of capillary blood from the upper arm of adults. The sample is shipped back to a laboratory for measurement of COVID-19 antibodies in blood.

KIT CONTENTS



Use by date



Consult instructions for use



This device contains sharps



Do not use if package is damaged



Alcohol Wipe

Cap for Collection Tube

Tasso Blood Collection Device

Return Pouch with Absorbent Pad


Instructions for Use



Store at 15-30°C



Prescription use only

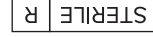

Sterilized by irradiation



Catalog number



Lot number



For in vitro diagnostic use