


TASSO-SST

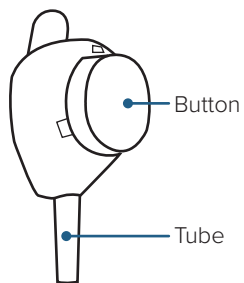


VIDEO

FIRST TIME USERS
Point your phone's camera here to watch:




or go to
www.tassoinc.com/tasso-sst-video


PREPARE

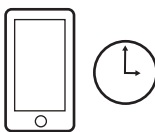


1. Get a timer.
You'll need it in step 9.

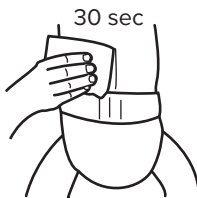

2. Hold a cloth warmed under hot water to the side of the lower back just above the diaper line for 30 seconds.

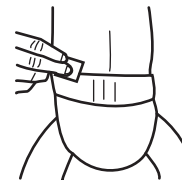


3. Clean warmed area with alcohol pad.

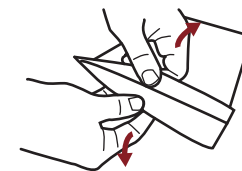


4. Open device pouch by pulling apart white and clear layers.


COLLECT

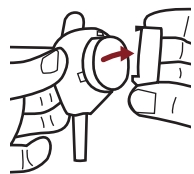


5. Remove clear plastic clover over the red button.

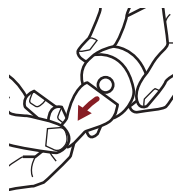


6. Peel paper tab behind the red button.
Keep tube pointing down.

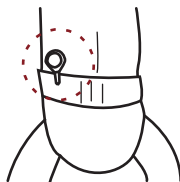

7. Stick device to lower back.
Do not remove once it's on.

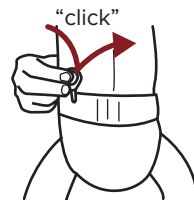

8. Press button quickly and firmly until it can't go any farther. Wait two seconds then let go.

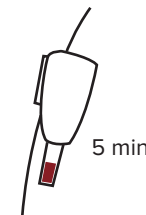

9. Start a five minute timer.
Keep child upright. You won't see blood right away.




10. After five minutes, peel off device.
No need to fill the tube completely.

RETURN




11. Remove tube by firmly twisting a quarter turn and pulling down.
This may take a bit of finger strength.

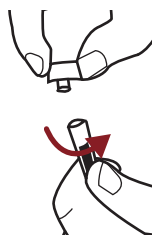

12. Twist cap fully onto the tube.




13. Place tube in the clear return bag and seal.
Leave the absorbent pad in the bag.

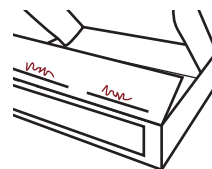

14. Write collection date and time on the inside flap of the box.



15. Place bag in box and seal with the strip.

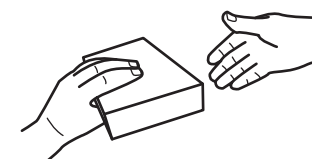

16. Return box within 24 hours of packing.
Throw out used Button in standard household trash.

For investigational use only.

**INSTRUCTIONS FOR USE
AGES 3 AND YOUNGER**




Tasso, Inc.
1631 15th Ave. W., Suite 105
Seattle, WA 98119



TASSO ONDEMAND

TASSO-SST



 Contact us for help:
1-408-708-8451

PN0101-03, A1

WARNINGS

Prescription use only.
Use before the expiration date affixed to the box.

One-time use only.

Keep out of reach of children.

Minor bruising may occur around the sample collection site.

For external use only.

Intended for self-use or use by a healthcare provider.

Use while seated as fainting may occur with any blood sampling procedure.




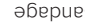

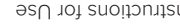
INTENDED USE

TASSO-SST is a single use blood collection device that is intended for the self-collection of capillary blood from people ages 3 months and older. The sample is shipped back to a laboratory for analysis, where it is intended to be centrifuged upon receipt for IgG and IgM serology measurements.

KIT CONTENTS

-  Use by date
-  Consult instructions for use
-  This device contains sharps
-  Do not use if package is damaged
-  Single use only
-  Store at 15-30°C
-  Prescription use only
-  Sterilized by irradiation
-  Catalog number
-  Lot number
-  For in vitro diagnostic use

SYMBOLS

-  Tasso Blood Collection Device
-  Cap for Collection Tube
-  Alcohol Wipe
-  Bandage
-  Return Pouch with Absorbent Pad
-  Instructions for Use