## **HOW TO SCAN**

## When you roll the case cart into the room

**Open Censitrac Prod -**

**SCAN -** Badge to log in then, <u>ROOM LOCATION</u> then, <u>CASE</u> CART

then,

**CLICK - SEND SCANNED DATA then, MENU** 

## YOU'RE DONE!

## If you bring in additional instruments or pans Open Censitrac Prod -

SELECT - WORK FLOW then, CASE TRACKING

**SCAN - CENSITRAC BARCODE** on the pick sheet then, the ROOM LOCATION then, INSTRUMENT PAN

then,

**CLICK – SEND SCANNED DATA** then, MENU

YOU'RE DONE!
THANKS FOR SCANNING!