


Mepilex® Border Sacrum & Heel Placement Guide

CORRECT SACRAL PLACEMENT

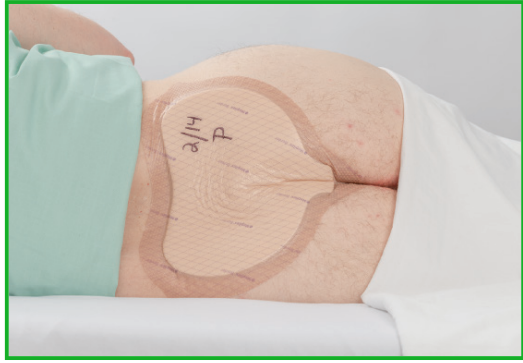

For more information on correct **Sacral** dressing application scan the code.

- ✓ Foam pad covers coccyx & sacrum
- ✓ Sealed edges
- ✓ Border flat and intact
- ✓ Entire dressing touching skin


CORRECT HEEL PLACEMENT



For more information on correct **Heel** dressing application scan the code.

- ✓ Foam covers heel and malleolus
- ✓ Entire dressing in contact with skin
- ✓ Assessment tabs accessible

INCORRECT SACRAL PLACEMENT

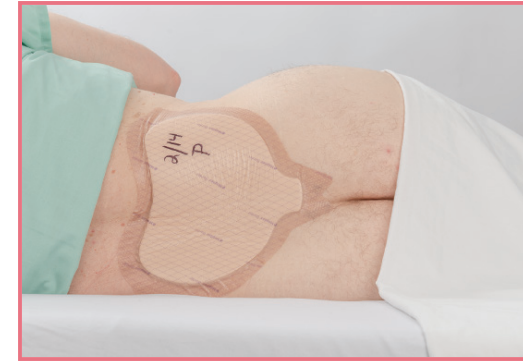

- ✗ Foam not covering coccyx
- ✗ Edges not sealed
- ✗ Dressing is too high/or too low

INCORRECT HEEL PLACEMENT



- ✗ Foam not covering entire heel and malleolus
- ✗ Edges not sealed
- ✗ Dressing is upside down & too high/or too low