

Dispose of Liquid Controlled Medications in the PharmaLock

Squirt or pour liquid pharmaceutical waste from syringes, vials, or IVs into the PharmaLock.




Change the PharmaLock Cartridges

Change the cartridge after 90 days, or when it gets full.

Remove the Cartridge:



1 Remove the lid from the PharmaLock.

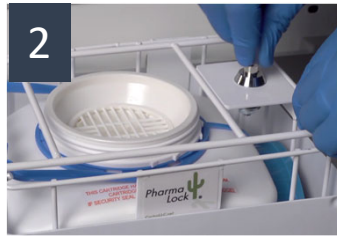

2 Use the key to open the cage.



3 Put cap on the cartridge. Turn clockwise to tighten. Several clicks tells you it is secure.



4 Pick up the cartridge by the handle. Discard in regular trash.

Order more cartridges: **PMM# 146807**


Replace the Cartridge:



1 Place new cartridge in cage with FRONT facing outward. Put cap in cage on right side of cartridge.



2 Close and lock the top of the cage. Remove key from the lock.

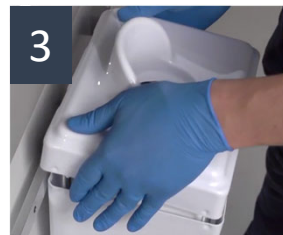

3 Securely place the lid on the container so splash guard faces out, towards the label.



4 Write the installation date on a piece of tape and stick it on the PharmaLock lid.



View PharmaLock video instructions.

Get More Information:
www.vumc.org/safety/waste/controlled-drug

